

USING THE MAGIC MOBILITY JACK

September, 2024

Instructions

- Remove the occupant from the wheelchair
- Find a level surface on which to work. The Jack works best and is safest when you use it on flat, even ground
- Before using the Jack, ensure the drive wheels are engaged
- Position the Jack underneath the wheelchair as shown in the pictures on the following page, ensuring that as much as possible of the Jack's lifting plate surface, is in contact with the lifting surface
- Raise the wheelchair by lowering the Jack. Exert some forward pressure in addition to the downward pressure
- Before working on the wheelchair ensure the chair is fully on the Jack.
- **Do not** put any part of your body underneath the wheelchair whilst it's on the Jack. In the case that the wheelchair falls, it could cause serious injury
- Lower the wheelchair slowly by lifting the end of the Jack, carefully keeping hold of the handle in case the wheelchair drops suddenly.

Placement of Jack for each Magic Mobility powered wheelchair

Magic 360



- Place Jack under the front bogie arm pivot

Frontier V4 RWD



- Place Jack under battery box

Frontier V6



- Place Jack under battery box

Frontier V4 FWD



- Place Jack under battery box

XT2



- Place Jack towards the front of the battery box

XT4



- Place Jack towards the front of the battery box

Extreme X8



- Turn the front wheels away from the Jack

Contact Magic Mobility if you require further information

3 International Court, Scoresby Vic 3179

Tel +61 3 8791 5600

Email sales@magicmobility.com.au

<http://www.magicmobility.com.au>